



People.Health.Care.

Debrisoft® Duo

Prepare Wounds
to **Heal**



2-in-1 debridement pad



Debrisoft® Duo supports wound healing for these patients



Before

After
Debrisoft® Duo



Before

After
Debrisoft® Duo



Before

After
Debrisoft® Duo

Sloughy Diabetic
Foot Ulcers

Venous Leg
Ulcers

Thick Necrotic
Eschar

Download the full
case study [here](#)

Download the full
case study [here](#)

Download the full
case study [here](#)

How to use Debrisoft® Duo



1 Moisten

Moisten in packaging with water or saline.

 See droplets icons



2 Adapt to patients needs

Start with either side, switch when needed.



3 Debride

Gentle pressure with long strokes or circular motion.



4 Dispose

Single use, dispose per local guidelines.



Cleanse the wound before using **Debrisoft® Duo** and again after, wash off emollients before use.



Do not oversaturate.

Click the link to download the **Debrisoft® Duo** User Guide




Click here



The New **Debrisoft® Duo** and Lolly Family



Debrisoft® Duo
10 x 10cm

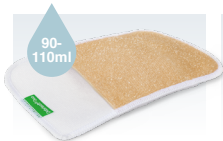

Debrisoft® Duo
13 x 20cm



Debrisoft® Lolly



Soft White Side

Removes loose slough,
exudate and biofilm



Textured Beige Side

Loosens firmly
adherent tissue

Click here

to find out more
about the new
Debrisoft® Duo
and Lolly family