



People.Health.Care.

- **Vliwasorb® Pro**
(Vlee-Va-Zorb Pro)
- Highly conformable dressing for highly exuding wounds



- **The advantages...**



NEW ergonomic shape, flexible design and softer absorbent core: Greater comfort and conformability



Very high absorbency and retention capacity: Protecting against maceration and extending intervals between dressing changes



Proteases and microbes are bound in the superabsorbent polymer core: Reducing microbial burden and odour



Protection and safety: Supporting atraumatic dressing changes and protection from leaks to improve the patient experience



Indications

Superficial wounds with moderate to very high levels of exudate, including:



Chronic wounds e.g. leg ulcers, pressure ulcers, diabetic ulcers, oncological wounds



Acute wounds e.g. partial thickness burns, postoperative wounds, trauma wounds, skin graft donor sites




How to use Vliwasorb® Pro

1. Cleanse the wound as per normal protocol and pat the edges dry - consider Debrisofting™ to remove barriers to healing.
2. Select the most appropriate shape and size of Vliwasorb® Pro, following assessment of the area to which this will be applied. E.g. if the patient has a leg ulcer on the gaiter region, a rectangular dressing applied up the limb may be most suitable.
3. Apply the white side of Vliwasorb® Pro to the wound.
4. In the case of deep wounds (e.g. laparotomy wounds), an appropriate dressing may be laid into the cavity first (e.g. Suprasorb® Ag rope) before applying Vliwasorb® Pro, depending on the wound status.
5. Ensure appropriate retention of Vliwasorb® Pro to secure the dressing against the wound. Compression systems (e.g. Actico®) may be used. If compression is not required/appropriate then an effective retention bandage should be used.
6. Vliwasorb® Pro should be changed when the absorbent capacity of the dressing has been reached. This is dependent upon the individual wound condition.

Precautions

- ✗ Do not use Vliwasorb® Pro on patients sensitive to any component of the dressing.
- ✗ Do not use Vliwasorb® Pro in cavity wounds, as the dressing will expand upon absorption of exudate.
- ✗ Do not use Vliwasorb® Pro on lightly or non-exuding wounds.
- ✗ Do not cut or tear Vliwasorb® Pro.
- ✗ It is not necessary to use a contact layer underneath Vliwasorb® Pro, as Vliwasorb® Pro has an integrated wound contact layer.

Tips

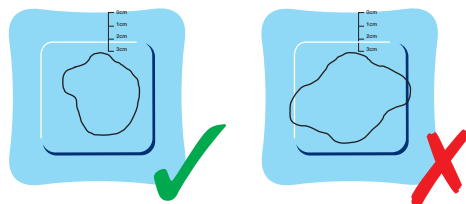

The size of the dressing should be based on the wound area; the dressing should overlap the wound by approximately 3 cm, ensuring the wound is entirely covered by the superabsorbent wound pad.

Ordering information

Vliwasorb® Pro

Sterile, individually sealed

Size (cm)	Wound pad (cm)	REF	Pip Code	Hospital Code	Pack contains	Size of equivalent Flivasorb® (cm) *
12.5 x 12.5	8.5 x 8.5	32 641	399-9893	EJA218	10 singles	10 x 10
12.5 x 22.5	8.5 x 18.5	32 642	399-9901	EJA219	10 singles	10 x 20
22 x 22	18 x 18	32 643	399-9919	EJA220	10 singles	20 x 20
22 x 32	18 x 28	32 644	399-9927	EJA221	10 singles	20 x 30

* The wider, more conformable border of Vliwasorb® Pro means the overall size of the dressing has increased from the sizes Flivasorb® was available in, without effecting the size of the wound pad.