


Suprasorb® P sensitive silicone foam dressing

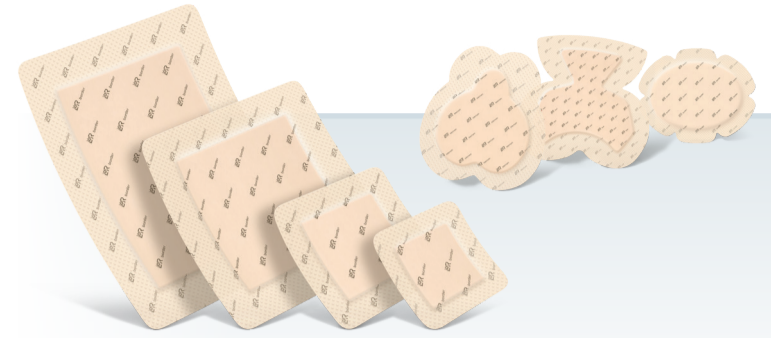

The right shape for every wound

Easy application and Removal

1. Prepare



2. Apply



3. Smooth



4. Remove



CLINICAL TOP TIP: For ease of removal rub a piece of tape onto one corner of the dressing then gently lift the edge.



Shoulder

The **Suprasorb® P sensitive heel dressing** conforms to the area, offering comfort and protection.



Back

The shape and conformability of the **Suprasorb® P sensitive heel dressing** fits comfortably on any wound on the back.



Foot

The **Suprasorb® P sensitive heel dressing** can easily be secured around the foot, conforming comfortably to its shape. Its flexible design allows for easy movement while staying firmly in place.



Knee

The **Suprasorb® P sensitive multisite dressing** keeps its shape while allowing complete flexibility and movement of the knee joint.



Elbow

The **Suprasorb P sensitive multisite dressing** is ideal for covering prominences like the elbow, keeping the joint flexible while the silicone is gentle on the skin.



Finger

The **Suprasorb® P sensitive non border 5cm x 5cm dressing** can be comfortably wrapped around the finger, providing a secure and snug fit. (see finger application)

Finger application



1. Remove the dressing from packaging and lay silicone side up.



2. Fold the top of the dressing over to cover the fingertip



3. Fold the dressing across both sides, taking time to smooth down the dressing



4. Secure in place. The two way stretch in the dressing allows the finger to bend and move naturally

COMPLEMENTS ALL SKIN TONES



Armpit

The **Suprasorb® P sensitive heel dressing** conforms securely under the axilla, remaining in place.



Toes

The **Suprasorb® P sensitive non border** can be easily cut to fit in between the toes. Fold the dressing in half, make two small cuts, open back out and slip over the toes. Tape down the edges of the dressing to secure in place. This provides a comfortable and conformable fit delivering protection to delicate hard to treat areas.



Back of the knee

The dressing flexibility and shape contours efficiently to difficult to treat areas, remaining securely in place.



Throat

The multisite dressing shape is suitable for a wide variety of locations.

Suprasorb® P sensitive is available in a full range of shapes and sizes for every wound type



Call our Customer Services Team: **08450 606 707**
or visit our website for more information about
Suprasorb® P Sensitive at: www.lohmann-rauscher.co.uk



Suprasorb® P sensitive order codes

Suprasorb® P sensitive border foam (with super-absorbent core)



Size dressing (cm)	Size wound pad (cm)	PIP	NHSSC	Pack size
7.5 x 8.5	4.5 x 5	414-6734	ELA1302	10
10 x 10	6.2 x 6.2	414-3640	ELA1282	10
12.5x12.5	8.5x8.5	414-3657	ELA1281	10
15 x 15	10.5 x 10.5	414-3665	ELA1298	10
20 x 20	15 x 15	414-6072	ELA1295	10
Sacrum 17 x 17.5	12 x 12.5	414-6213	ELA1299	10
Sacrum 23 x 23	16.5 x 17	414-6221	ELA1288	10
Heel 23.5 x 25	17.5 x 19	414-3673	ELA1293	10
Multisite 12 x 15	7 x 10	414-6239	ELA1284	10

Suprasorb® P sensitive

non-border foam (without super-absorbent core)



Size dressing (cm)	Size wound pad (cm)	PIP	NHSSC	Pack size
5 x 5	5 x 5	414-6148	ELA1285	10
10 x 10	10 x 10	414-6155	ELA1289	10
10 x 20	10 x 20	414-6163	ELA1296	10
15 x 15	15 x 15	414-6171	ELA1286	10
20 x 20	20 x 20	414-6189	ELA1283	10
7.5 x 7.5	7.5 x 7.5	414-6197	ELA1301	10
15 x 20	15 x 20	414-6205	ELA1290	10