



People.Health.Care.

When I want to be sure, I *debrisoft* it.

Discover the new side of Debridement with **Debrisoft® Duo**

### Efficient 2-in-1 debridement pad:

- The **original soft, white** side removes debris, slough, exudate and biofilm from the wound
- The **new textured beige** side makes it easy to loosen firmly adherent, fibrinous devitalised tissue such as slough, skin flakes and keratoses.<sup>1</sup>



### Top Tip

To moisten Debrisoft Duo use the packaging with water or saline



### Debrisoft® Lolly

- For the debridement of deep wounds and hard-to-reach areas
- Secure fixation of the fibre head
- Ideal for surgical invasive wounds
- Ideal for cavities and in between toes



### Debrisoft® Duo 10 x 10 cm

- Designed for debridement of fibrinous slough and challenging wounds
- The textured beige side makes it easy to loosen firmly adherent fibrinous slough
- The soft white side removes debris, slough, exudate and biofilm from the wound
- Flexibility to use either side of the pad dependent on the wound condition and patient need



### Debrisoft® Duo 13 x 20 cm



# Tips for Effective Use

## Moisten Debrisoft® Duo with amount indicated

- ✓ Wash off any emollients before using Debrisoft® Duo
- ✓ Debrisoft® Duo is for single use only
- ✓ Following the use of Debrisoft® Duo, reassess the wound/skin before selecting an appropriate dressing/therapy
- ✓ Use Debrisoft® Duo for at least 2-4 minutes
- ✓ Larger areas may take longer to treat, depending on the amount of slough, debris or hyperkeratosis present
- ✓ When addressing biofilm, Debrisoft® Duo should be used regularly at each dressing change. Consider using in combination with an antimicrobial dressing such as Suprasorb® Liquacel Ag (follow local guidelines)

# Key Considerations

## Do not oversaturate

- ✗ Debrisoft® Duo must not be used as a wound dressing
- ✗ Do not cut Debrisoft® Duo
- ✗ Debrisoft® Duo should not be used if there is a known sensitivity to any of the components of the product

1. In vitro data on file C.Panhözl in R

Debrisoft® is a registered trademark of L&R.



# Debrisoft® Duo - Step by Step.

Before using Debrisoft, wash off any emollients (moisturisers), creams or ointments.



## 1. Moisten

Debrisoft® Duo should be moistened before usage with water or any common wound irrigation solution. See drop icon for fluid amounts.



## 2. Adapt to needs

Start with either side of Debrisoft® Duo, depending on the wound condition and switch as needed.



## 3. Debride

Applying gentle pressure using long sweeping strokes or circular motion, wipe the pad over the wound or the affected skin.



## 4. Done

Dispose Debrisoft® Duo product after use (Follow local guidelines).



**Now available on NHS Supply Chain, off script ordering systems and coming soon on Drug Tariff.**



Upgrade to  
**Debrisoft® Duo** today,  
contact us here...



For further  
information on  
**Debrisoft® Duo**  
scan the QR code



5 037696 712783 >

M3546 V1.1

For further guidance please call our  
Customer Services Team on: **08450 606707**  
or visit: **www.Lohmann-Rauscher.co.uk**